Walking Sticks (incl. Extendable)

Product	Type
VP155	Ergonomic walking stick left hand
VP156	Ergonomic walking stick right hand

Max. User Weight: 100 kg (16st.)

Product	Type
VP155B	Ergonomic 2 section walking stick left hand
VP155D	Ergonomic 2 section walking stick right hand
VP155F	Walking stick with wooden handle
VP155K	Extendable wooden handled walking stick

Max. User Weight: 120 kg (19st.)

Ergonomic Walking Stick
VP158ELL/ELS/ELM/ERS/ERM/ERL

Aluminium Walking Stick
VP158S/M/L

Max. User Weight: 127 kg (20st.)

DO NOT exceed the weight limit stated – doing so could put the user at risk.

NB. These are not medical devices.

Usage and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the walking stick.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.
- If you have purchased a folding/collapsible walking stick, check that it is properly and securely clipped together.
- With some models it is possible to adjust the height of the walking stick. In this case, where possible, please consult a doctor, physiotherapist or your retail outlet to check the correct height for you.

INTENDED USE

The walking stick is designed to support the user whilst walking.

HEIGHT ADJUSTMENT (IF APPLICABLE)

1. Grip stick shaft with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft.
2. Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position.
3. **Prior to use always ensure that the locking pin fully protrudes from the selected hole position and by applying downward pressure on the handle of the walking stick test that the walking stick has been assembled correctly.**

CLEANING

Clean your Walking Stick using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors/rivets
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)